sigvaris

magnide[®] on/off

DE	Gebrauchsanweisung	2
EN	How to use	4
FR	Utilisation	6
IT	Utilizzo	8
NL	Gebruiksaanwijzing	10
DA	Brugsanvisning	12
NO	Hvordan bruke	14
SV	Bruksanvisning	16
FI	Käyttöohje	18
ES	Instrucciones de uso	20
PT	Mode de utilização	22
PL	Instrukcja użytkowania	24
JP	使い方	26

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ON

1

2

3

4

5

OFF

1

2

3

4

ON

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4

ON

1. Place the magnide on/off across the foot (ensure that the white flap near the toes is well folded back, so that the magnets connect)
2. Pull the stocking over the magnide on/off
3. Slide the stocking over the foot, ensuring that the heel of the stocking sits in the right place
4. Make sure that the dark blue loop sticks out above the stocking so it can be gripped
5. Pull the dark blue loop to remove the magnide on/off from the stocking

OFF

1. Slide the magnide on/off over the stocking
2. Firmly pull the dark blue fastener so that the magnide on/off is allocated just above the ankle
3. Fold the stocking over the magnide on/off as far as possible
4. With small movements, starting at the heel, slide the stocking and magnide on/off over the foot from the heel

Preparation for re-use

- I Fix the dark blue loop to the top of the magnide on/off using the magnet. Make sure that the dark blue magnet surfaces are connected to each other and that the magnet closes
- II Turn the magnide on/off over and place it on a flat surface
- III Fold both the sides with magnets to the middle. Make sure that the dark blue magnet surfaces are connected to each other and that the magnets close. Now only the dark blue stitching should be visible on the outside
- IV Fold the white flap (near the toes) upwards so that the magnet closes

SIZING

Size	Shoe size	
Medium	≤ 38	XL also suitable for extra wide feet with cA up to 28 cm 
Large	39–43	
X-Large	≥ 44	

INTRODUCTION

magnide on/off provides an effective solution for putting compression stockings on and taking off.

Your new magnide on/off is characterised by a high level of effectiveness, comfort and quality. The magnide on/off has been designed to eliminate problems that can occur when putting stockings on or taking them off, such as damaged and overstretched stockings.

magnide on/off is a valuable aid not just for those who wear compression stockings but also for their professional caregivers as well.

To be sure you get the most out of the magnide on/off read the enclosed product information carefully. If you have any questions, please contact your supplier.

PRODUCT BENEFITS / CHARACTERISTICS

- Reduction of physical strain for both caregiver and stocking wearer
- Increase patient's sense of independence
- Increase in therapy compliance
- Stockings retain therapeutic compression longer
- The magnide on/off is made of innovative, smooth and light material

Use a separate magnide on/off for each patient. If the magnide on/off needs to be used on more than one patient, you must cleanse the product with a 70% ethanol solution.

WASHING INSTRUCTIONS

- magnide on/off may only be washed by hand, at a low temperature (30°C)
- Dry the product with a towel - never iron
- Do not dry clean or treat with bleach
- Do not expose the product to sources of heat like bright sunlight, heaters or central heating

IMPORTANT

- Warning! Do not step on the magnide on/off. The highly slippery material makes it a slipping hazard.
- Warning! The magnide on/off contains several magnets. For this reason, keep this product away from pacemakers and defibrillator. The magnetic field may interfere with the proper operation of this equipment (minimum distance 30 cm).
- When using the magnide on/off it is important to use the correct size for you. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly. The correct size is determined by your shoe size.
- Wear special rubber gloves when using the magnide on/off. This gives you maximum grip on the stocking, and reduces the physical strength needed.
- This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or construction. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. To use this service you need to contact your magnide on/off supplier. You are asked to return the product washed and to follow the washing instructions.

MD Medical Device

Intended purpose: Facilitates donning and doffing of medical compression stockings.

Notice: All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

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SIGVARIS and SIGVARIS GROUP are registered trademarks of SIGVARIS AG, CH-9014 St. Gallen/Switzerland, in many countries worldwide.
© 2020 Copyright by SIGVARIS AG, CH-9014 St. Gallen /Switzerland