sigvaris

**sim-slide®**

<b>DE</b>	Gebrauchsanweisung	2
<b>EN</b>	How to use	4
<b>FR</b>	Utilisation	6
<b>IT</b>	Utilizzo	8
<b>NL</b>	Gebruiksaanwijzing	10
<b>DA</b>	Brugsanvisning	12
<b>NO</b>	Hvordan bruke	14
<b>SV</b>	Bruksanvisning	16
<b>FI</b>	Käyttöohje	18
<b>ES</b>	Instrucciones de uso	20
<b>PT</b>	Mode de utilização	22
<b>PL</b>	Instrukcja użytkowania	24
<b>JP</b>	使い方	26

ON

1

2

3

4

5

OFF

1

2

3

4

ON

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4

ON

1. Place the sim-slide over the foot. To prevent the sim-slide from gliding off the foot, gently pull on the dark blue tabs.
2. Pull the stocking over the sim-slide
3. Slide the stocking over the foot, so that the heel of the stocking fits into the correct position
4. Place one hand firmly on the heel and with the other hand pull the loop on the sim-slide
5. Gradually pull the sim-slide out of the stocking

OFF

1. Slide the sim-slide over the stocking
2. Firmly pull on the dark blue tabs so that the sim-slide sits just above the ankle
3. Fold the stocking over the sim-slide as far as possible
4. Starting at the heel, gradually slide the stocking and the sim-slide from the foot

## SIZING

Size EU	Shoe size
Small	≤ 34 – 36
Medium	37 – 38,5
Large	39 – 41
X-Large	≥ 42

Size UK	Shoe size women	Shoe size man
Small	≤ 1½ – 3½	≤ 2½ – 4
Medium	4 – 5½	4½ – 6
Large	6 – 7	6½ – 7½
X-Large	≥ 7½	≥ 8

Size US	Shoe size women	Shoe size man
Small	≤ 4 – 6	3 – 4½
Medium	6½ – 8	5 – 6½
Large	8½ – 9½	7 – 8
X-Large	≥ 10	≥ 8½

## INTRODUCTION

sim-slide provides an effective solution for putting on, or taking off, compression stockings.

Your new sim-slide is characterised by a high level of effectiveness, comfort and quality. As you know, putting on compression stockings is an intensive activity which often causes problems such as damaged stockings or strained muscles. The sim-slide has been designed to address these problems.

sim-slide is a valuable aid not just for the stocking wearer but also for their professional carers.

To be sure you get the most out of the sim-slide read the enclosed product information carefully. If you have any questions, please contact your supplier.

## PRODUCT BENEFITS / CHARACTERISTICS

- Reduction of physical strain for both caregiver and wearer
- Increase in independence
- Increase in therapy compliance
- Stockings retain therapeutic compression longer
- sim-slide is fabricated from innovative smooth, light material

Use a separate sim-slide for each patient. If sim-slide needs to be used on more than one patient, you must clean the product with a 70% ethanol solution.

## WASHING INSTRUCTIONS

- sim-slide may only be washed by hand, at a low temperature (30 °C)
- Dry the sim-slide only with a towel – never iron
- Do not dry clean or treat with bleach
- Do not expose the sim-slide to sources of heat like sunlight, heaters or central heating.

## IMPORTANT

- Warning! Do not step on the sim-slide: The highly slippery material makes it a slipping hazard.
- When using sim-slide it is important to use the correct size for you. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly. The correct size is determined by your shoe size.
- Wear special rubber gloves when using sim-slide. This gives you maximum grip on the stocking, and reduces the physical strength needed.
- This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or construction. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. To use this service you need to contact your sim-slide supplier. You are asked to return the product washed and to follow the washing instructions.

**MD** Medical Device

**Intended purpose:** Facilitates donning and doffing of medical compression stockings.

**Notice:** All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

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Tailles	Pointures
Small	≤ 34 – 36
Medium	37 – 38,5
Large	39 – 41
X-Large	≥ 42

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SIGVARIS and SIGVARIS GROUP are registered trademarks of SIGVARIS AG, CH-9014 St. Gallen/Switzerland, in many countries worldwide.
© 2020 Copyright by SIGVARIS AG, CH-9014 St. Gallen /Switzerland